

Place the Tellus 5 on a flat surface using the built in stand or mount to a wheelchair. If you are using the Tellus 5 Intelligaze please attach the device to the mount.

To Turn ON:

Turn ON the Tellus 5 using the button located on the right side of the device.


Once Windows has loaded you will see on the desktop will be the icons for Mind Express 5



and The Grid



Open your preferred communication software. If you are setting up for a new user, you can connect an external keyboard and mouse if desired for editing. Follow The Grid or Mind Express manuals for further use and programming instructions.

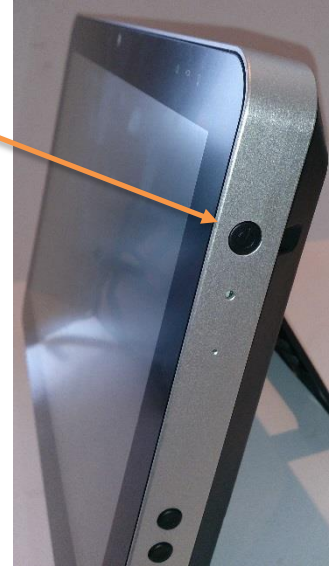

To Charge:

Use the charger socket on the back of the device (See the Tellus 5 guide for further information).



To Turn OFF:

Shut down the computer from the Windows START button as you would a laptop or computer and shut down or sleep as required. If you are using communication software you can also program a cell to shut down the device.

Using an external Switch to Turn On:

It is possible to set up the Tellus 5 so that an external switch can be used to turn the device on. Please see additional guides on how to configure these settings.

3G Sim Card:

The Tellus 5 has an internal SIM card slot that enables you to send SMS using either The Grid 2 or Mind Express 5. The SIM card modem can also be set up for 3G data for internet access on the move. Further instruction on setting up the SIM card can be found on additional instruction sheets.

Switches:

You can plug up to 2 switches into the Tellus 5 switch jacks that are on the back of the device. These can be set up to provide various types of scanning input to both Mind Express and The Grid 2. For further information please refer to the Grid 2 or Mind Express manual.



If you are using The Grid 2 there is also the option to set the user programmable buttons on the back of the Tellus 5 as switches.