

WoodPecker USB Manual

Content

WoodPecker Installation

WoodPecker Utilities:

- WoodTest
- WoodShow
- WoodMouse
- WoodKeys

WoodPecker installation

Follow these instructions to install the WoodPecker driver and utilities.

Download WoodPecker installer:

https://www.zytech.com.au/products/switches_and_connectors/woodpecker

Open "WoodPecker.exe" from your Downloads folder.

The Welcome dialog will pop up. Please click Next.

In the next screen, choose the type of WoodPecker interface (USB or Serial) you purchased and click Next.

Follow the rest of the installation instructions to finish the installation.

WoodPecker USB

Plug in your WoodPecker USB to a free USB port.

Windows operating systems have native support for the WoodPecker USB.

WoodTest



After installation, it is recommended that you start the WoodTest utility.
This can be done by going to **Start > Programs > WoodPecker > WoodTest**.

WoodTest Menu

Joystick mode	Set WoodTest to joystick mode. In this mode you can test whether your joystick is functioning properly.
Switches mode	Set WoodTest to switch mode. In this mode you can test whether your switches are functioning properly.
Select WoodPecker type	Choose USB (5 switch connections)
Query	<i>This is not relevant for WoodPecker USB</i>
Exit	Shutdown WoodTest

The first time you use WoodTest, you will be asked whether you want to test your switches or the joystick.
Make your choice and press **OK**.

Depending on your choice, the WoodTest screen will appear:

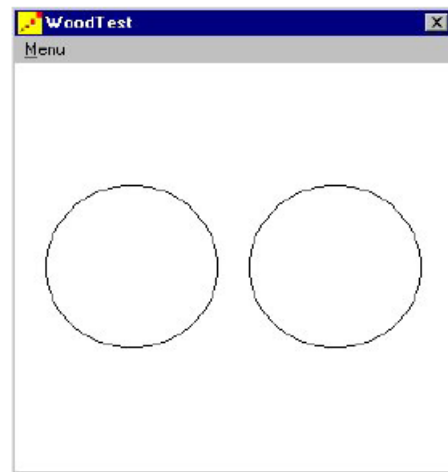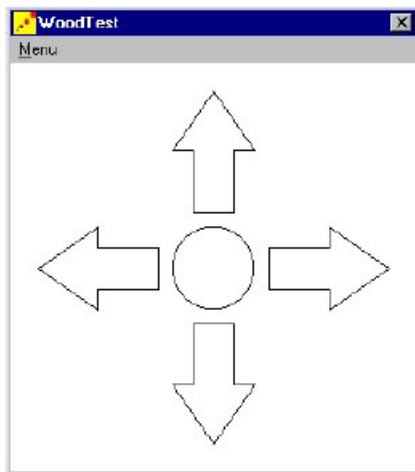

If WoodPecker is connected properly, the buttons or the arrows should light up if you either press a switch or move the joystick. If this is not the case, check the following:

- Make sure you selected the correct type Of WoodPecker (USB) by going to **Menu > Select Woodpecker type**.
- Make sure all cables are connected correctly and/or WoodPecker is connected properly to a USB port on your PC.

If the problem persists, please contact your local distributor or Jabbla.

WoodShow

With WoodShow, you can make action-reaction games or exercises. Pressing a switch can bring up a picture, play a sound or launch another program.

WoodShow menu

Switch Actions	Define the functionality of the switches and choose a profile.
Options	Set program options.
Select WoodPecker type	Choose USB (5 switch connections).
Full Screen	Set the WoodShow window to Full screen. Press Esc or left click to restore the window to its previous position and size.
Exit	Shutdown WoodShow.

To work with WoodShow, follow these steps:

1. Open a profile


Select **Menu > Switch actions**. Select the profile you want to work with in the list. Every profile can have a different definition for the switches.

e.g. suppose there are two profiles 'Animals' and 'Food'. If you press switch 1 while the profile Animals is active, a picture of a lion could appear. If you press switch 1 while Food is loaded, a picture of an apple could appear. The 'Default sample' profile is already included and can be used to experiment.

To make a new profile, press **New** and enter a name for the profile.

To delete a profile, select the profile you want to delete in the list and press the **Delete** button.

2. Define the functionality for the switches

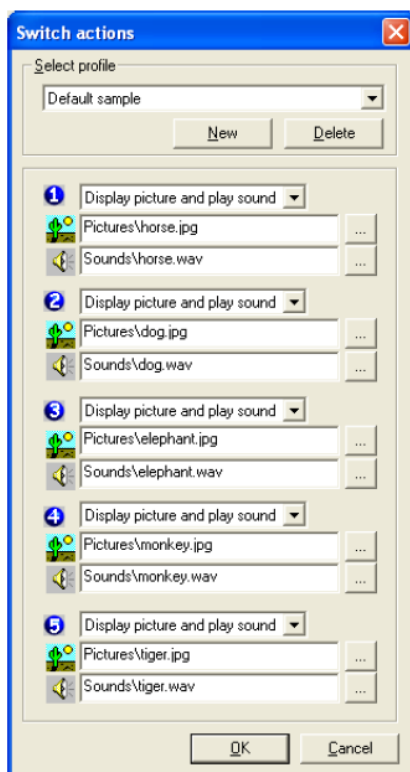

Do nothing	When this switch is pressed, nothing will happen.
Display picture	Displays a picture (GIF, JPG, BMP) in WoodShow. Load a picture by pressing the " ..." button.
Play sound	Plays a "WAVE" sound in WoodShow. Load a wave by pressing the " ..." button.
Execute a program	Executes the specified file. Define the file you want to execute by pressing the " ..." button. You can specify command line arguments by simply adding them to the text box.
Display picture and play sound	Displays a picture and plays a sound simultaneously when a switch is pressed.

3. Additional options

You can make a few additional settings by going to **Menu > Options**.

These settings are global settings, so they are not dependent on the profile:

Background color

Click Color to set another background colour.

Clear screen after displaying picture

If, due to a switch press, a picture was loaded into WoodShow, you can make it disappear automatically after a set time period. This time is reset if another switch is pressed.


Enable keyboard access

If you want to be able to use the keyboard to simulate switch presses, you should check this option. You can use the arrow keys and the space bar to accomplish this.

WoodMouse

With WoodMouse, a user can move the mouse cursor on the screen using a joystick connected to the Woodpecker interface (**or in the case of a Woodpecker USB, the 5 switches can also be used**). Moving the joystick in the desired direction will move the mouse cursor in that same direction. There are several settings that can be made to personalise WoodMouse.

To bring up the "Program options" window, right click the WoodMouse tab in the taskbar:

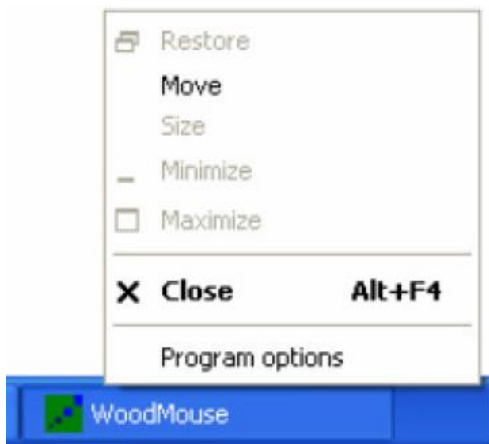


Note that WoodMouse can be shut down using this menu.

If you have selected the Use button bar option, the following window will appear:



Press the last button to bring up the Options window:

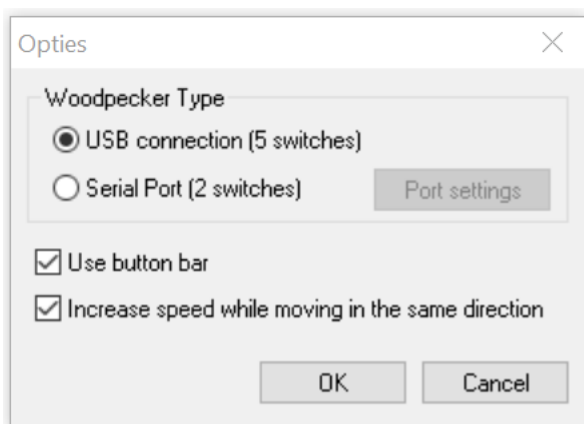

WoodPecker Type - Choose USB (5 switches).

Use button bar

There are two different ways to click with the joystick:

- No button bar: To click, press the fire button. To double click, press the fire button twice. To drag, hold the fire button and move the joystick in the desired direction. To stop dragging, release the fire button.
- Button bar: If double clicking or dragging is too hard (or if you want to right click), you simply move the mouse pointer to the desired WoodMouse button and press the fire button. After that, move the joystick to the desired spot and press the fire button again. WoodMouse will double click, right click or start dragging depending on the button you selected. In the case of dragging, if you want to stop dragging, press the fire button again.

Increase speed while moving in the same direction

This option indicates whether the joystick should move faster if you keep moving in the same direction. The longer you move in the same direction, the faster the movement will be.

How to operate WoodMouse using a WoodPecker USB with switches?

Switch 1: Pressing switch 1 will move the mouse pointer down for as long as the switch is held. Releasing the switch will stop the movement.

Switch 2: Pressing switch 2 will move the mouse pointer to the left for as long as the switch is held. Releasing the switch will stop the movement.

Switch 3: Pressing switch 3 will move the mouse pointer up for as long as the switch is held. Releasing the switch will stop the movement.

Switch 4: Pressing switch 4 will move the mouse pointer to the right for as long as the switch is held. Releasing the switch will stop the movement.

Switch 5: To click on the current position of the mouse pointer, press switch 5 once. To double click, press switch 5 twice.


WoodKey

WoodKey allows you to use your switches in other programs.

You can program your switches to do one of several possible actions:

- Send a key to a program
- Send a key combination (macro)
- Play a sound file (WAV)
- Perform a Windows function (like altering the volume)
- Send a command to DocReader, Skippy or Mind Express.

When you start WoodKey, the **Button definition** window will appear:

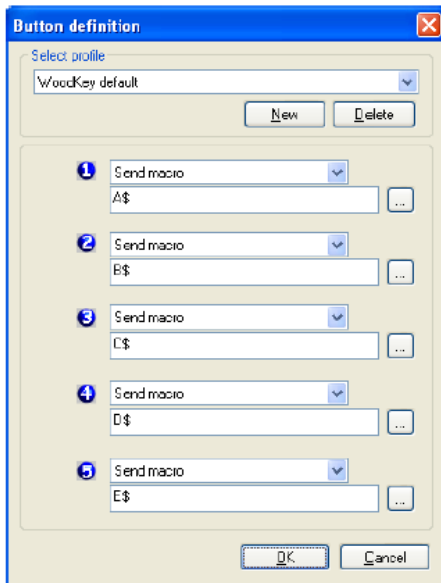


Programming the switches

In the Button definition window, you can program 5 switches. These relate to the 5 switches on the Woodpecker USB.

You can select an action in the list next to each button :

Send a macro	See Editing macros below. Start making a macro by pressing the button. Whenever you press the switch, this macro will be sent to the active application.
Send a key	Select a key in the list. Whenever you press the switch, this key will be sent to the active application.
Play Wave file	Select a Wave file using the “...” button. Whenever you press the switch, this sound file will be played.
Start program	Select an .exe program file using the “...” button. Whenever you press the switch, this program will be started.
Perform Windows function	Select an action in the list.
Send command to Mind Express	Select an action in the list. The Mind Express software has to be installed on this computer.
Send command to DocReader	Select an action in the list. The DocReader software has to be installed on this computer.
Send command to Skippy	Select an action in the list. The Skippy software has to be installed on this computer.



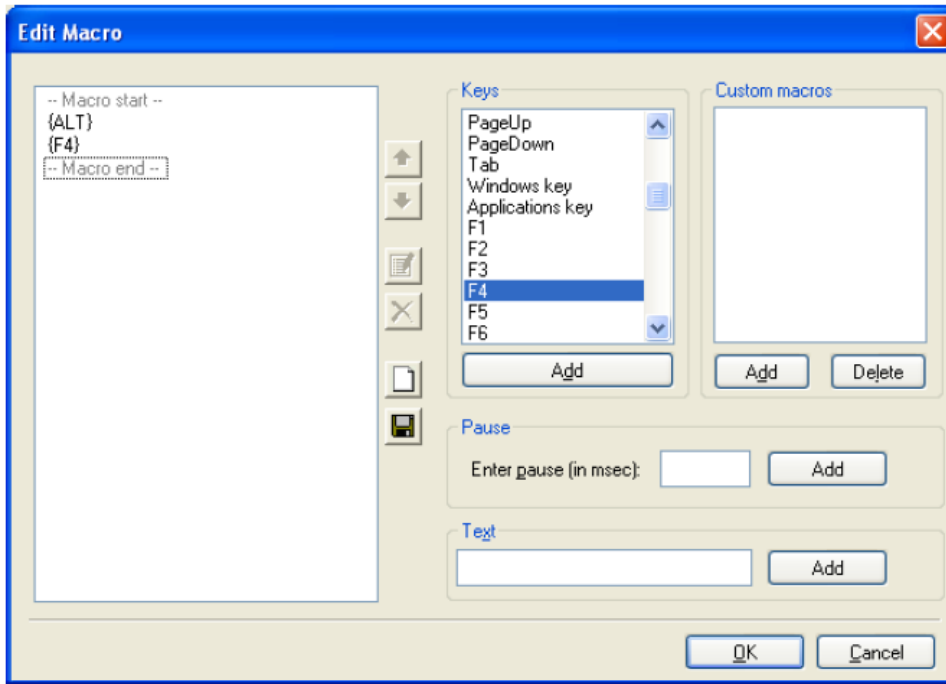



Editing macros



A macro is a programmable list of keystrokes. You can add different items to the list:

- Text or characters: type your character or your text in the **Text** edit box. Press **add** to add the text to the list.
- Keys: double click on a key in the **Keys** list to add a keystroke.
- Pause: type the amount of milliseconds you want to pause between commands in the list when executing a macro.
- Custom macros: when you make a list of commands, you can save that list (by pressing the save icon). When you do that, you'll be prompted to enter a name for this custom macro. After you have given it a name, it will appear in the custom macros list. When you double click on a custom macro, all commands in the custom macro will be added to the current command list.

Icons

	Moves the selected command up in the list.
	Moves the selected command down in the list.
	Opens the selected command for editing.
	Deletes the selected command.
	Removes all commands in the list and makes a new empty list.
	Saves the command list as a custom macro.

If you don't want a switch to perform an action, you can choose the action **Send a macro** and leave the edit box empty.

Different profiles

Most of the time, you will want to use different profiles for different applications. WoodKey allows you to make as many profiles as you like.

E.g. you could use a PowerPoint profile to send Page Up, Page Down so you can browse a Power-point presentation with your switches. And you could use a Media player profile to play and stop a movie file.

To make a new profile, press **New** and enter a name for the profile.

To delete a profile, select the profile you want to delete in the list and press the **Delete** button.