


One of the many benefits of the Tellus 4 and Mobi 2 is that they can be turned on by pressing an external switch. This is very useful for someone who may have difficulty using the tablet's in built on /off switch.

To enable the device to be turned on with a switch you will need to enable it in the Jabbla tools by following the steps below:

1. The Jabbla tools program is located in the Windows desktop tray. Using a mouse or stylus right click on the  icon to bring up the menu below.

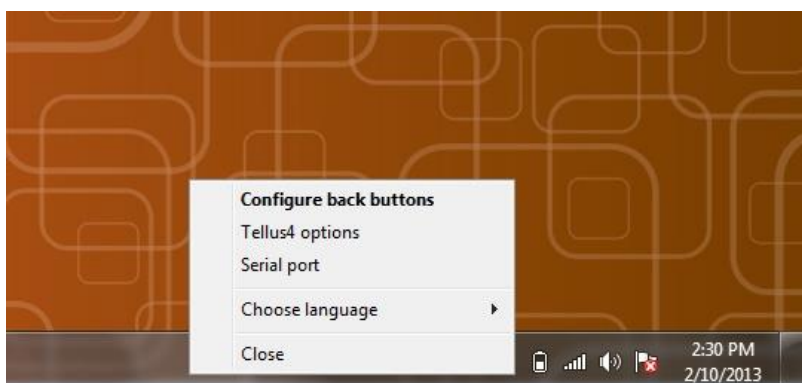

2. Select **Tellus4 options** and the following dialogue box will open.



3. Next to **Switch 1 starts device** select the amount of time that the switch should be pressed before the device turns on. In our experience 2 seconds is a good length of time as it is not too difficult for most to press the switch for that amount of time and also helps prevent accidental presses. Of course, you can select the length of time that suits you.
4. Select **OK**