

Switch 1: Advances the selection. (Move highlight)


Switch 2: Activates the selection. (Choose highlighted)

To scan the user repeatedly clicks SWITCH 1 until they reach the desired grid cell, then clicks SWITCH 2 once to select the grid cell.

- Any scanning pattern can be used – whichever is most suitable for the user’s requirements.
- No timing skills are required which can be very beneficial for users with movement difficulties and prefer not to hit a switch in a timed manner.

Setting up Two Switch Step Scanning in Mind Express 4

1. In Mind Express select **Tools** from the top menu bar. Select **Scanning Selection** from the drop down menu.

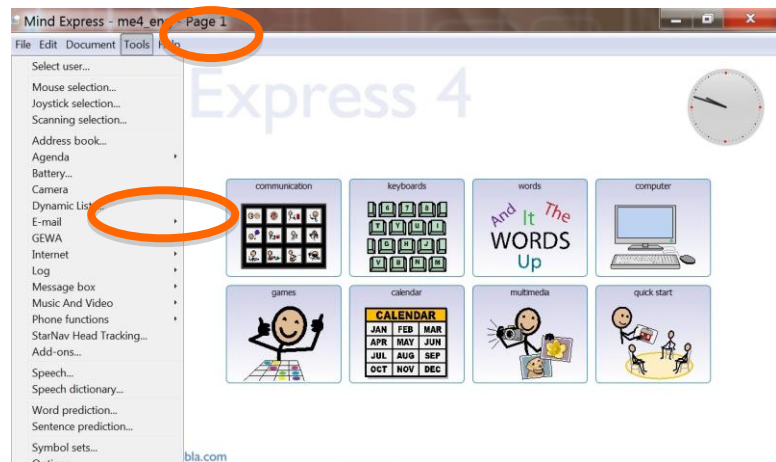

2. In the Scanning selection pop up window set **Advance** to “Tap Switch” and **Select** to “Tap switch 2”

The remainder of the settings are user specific. Set either accept on press (click of switch) or on release (release of switch) depending on what the user prefers for their switching method. Different scanning patterns can be chosen from the Method menu such as row column or linear scan.

