

If you are having problems getting your AUDITORY SCANNING this may be the setting you need:

- 1. Click the MENU bar to expand the options at the top > choose SETTINGS
- 2. Select ACCESS



3. Select SWITCHES – click on (or touch) the icon of the switch)



## 4. Select ACTIVATION



5. Scroll down to see further settings (this is the trick you may have missed!)

Ĵ			
er nt control	Advance Scan rows, then cells   Control Contro	Activate Switch 1 Activate after 2 seconds Activate after 2 seconds Off Swap advance and activate switches during scan Off Automatically start scan Off Repeat previous selection with activate switch Off Scan backwards before activating Advance backwards every 1.5 seconds +	

ZYTEQ pty ltd PO Box 190 South Melbourne Victoria 3205Ph:03 9696 2944 Fax:03 9696 1755 Freecall:1800 818 353Email:info@zyteq.com.auWebsite:www.zyteq.com.auA.B.N. 81132 895 268

6. Select the drop-down options under HIGHLIGHTING

		Advance backwards every 1.5 seconds	
ontrol	Highlighting Colour cell background No audio highlighting Off Play sound when activated On Gray out cells not part of current row, column, or scan block	Further options         Skip over empty cells         Reverse scan direction         None         Back up one level         None         Stop scan         None         Maximum number of scans         3	

٠

## 7. Choose Speak Audio Description



SELECT OK